

OCUVANE[®] BABY

EYELID WIPES

MEDICAL DEVICE

Sore, red, rheumy eyes in children are often caused by narrowing of the tear ducts. Excessive tear production as a result of a blockage leads to the build-up of various pathogens on the surface of the eye, leading to inflammation with a rheumy secretion.

OCUVANE[®] BABY eyelid wipes gently clean and nourish the eyelids and effectively alleviate inflammatory symptoms. Due to its vitamin E content, it has soothing, moisturizing, and regenerative effect.

For children's sensitive skin, a hypoallergenic, tear-free formula has been developed. It is free of Paraben, coloring agents and added fragrance, with a neutral pH. Suitable for use from the age of 3 months.

Applications

- For periocular cleansing of rheumy eyes
- To alleviate blocked tear duct symptoms

Ingredients


Disodium cocoamphodiacetate, phenoxyethanol, ethylhexylglycerin, polyhexamethylene biguanide, tocopheryl-acetate (vitamin E), polysorbate 20, citric acid, purified water

Instructions for use

Before use, wash your hands, then open the sachet and remove the impregnated **OCUVANE[®] BABY** eyelid wipes.

1. To cleanse of eyelids and eyelashes

In mild cases, apply once daily; in cases of severe discharge, apply twice daily, in the morning and evening, to remove rheumy secretions from around the eyelids.

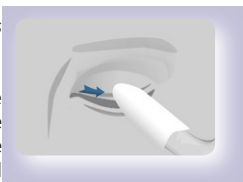

⚠️ Precautions and warnings

For external use only. Do not use the wipe after the expiry date stated on the packaging, or if the packaging is damaged. Very rarely, hypersensitive reactions to the ingredients of the product may occur, with symptoms such as a burning sensation, itching, tearing, redness and swelling of the eyelids. In such cases the use of the wipe should be discontinued. Although the symptoms outlined above will disappear within a short period of time, in such cases continued use of the wipes is not recommended. If, despite treatment, you experience increased inflammatory symptoms (reddening, tearing, rheumy discharge, or pain), consult an ophthalmologist immediately.



Store at room temperature. Keep out of the reach and sight of small children.



SolMed Pharma Kft.

19 Ady Endre Street, 1024 Budapest
Hungary
Email: info@solmedpharma.hu

www.ocuvane.com