

# OCUVANE®

## EYELID WIPES

### MEDICAL DEVICE

Tired, irritated, red eyes can be caused by the accumulation of dirt and allergens on the eyelid, by wearing contact lenses, and through prolonged use of monitors, which can lead to tear production disorders. Regular cleansing and care of the eyelids is essential for long lasting alleviation of complaints by restoring the function of the tear glands in the eyelids.

The special ingredients of the **OCUVANE®** eyelid wipes effectively remove the dirt and pollen that have accumulated along the edge of the eyelid. Daily use eliminates the blockage of the tear glands in the eyelid, restoring healthy tear production. The wipes' special ingredients have antiseptic, anti-inflammatory and regenerative effects, improving eye and contact lens comfort.

#### Applications

- To alleviate the symptoms of bacterial and allergic conjunctivitis
- For dry eye complaints
- Decrease eye fatigue
- To improve the comfort of wearing contact lenses
- Before and after eye surgery, to clean the eye area

#### Ingredients

Disodium cocoamphodiacetate, phenoxyethanol, ethylhexylglycerin, polyhexamethylene biguanide, tocopheryl-acetate (vitamin E), polysorbate 20, citric acid, purified water

#### Instructions for use

Before use, wash your hands, then open the sachet and remove the impregnated **OCUVANE®** eyelid wipes.

#### 1. To cleanse eyelids and eyelashes

As shown in the figure, wrap the wipe around your index finger; then gently wipe the edge of the eyelid with long strokes, several times.



#### 2. Cleansing the edge of the eyelid



In front of a mirror, with fingers placed on the bottom eyelash, invert and hold open the lower eyelid. With the wipe wrapped around your index finger, gently wipe the edge of the eyelid in long strokes, several times. Be careful not to touch the eyeball. Using a fresh wipe, treat the other eye.

In order to remove accumulated dirt from the edges of the eyelids, in mild cases, apply once daily in the evening; in more severe cases, apply twice daily in the morning and evening.

#### ⚠ Precautions and warnings

Do not use the wipe after the expiry date stated on the packaging, or if the packaging is damaged. Very rarely, hypersensitive reactions to the product's ingredients may occur, with symptoms such as a burning sensation, itching, tearing, redness and swelling of the eyelids. In such cases use of the wipe should be discontinued. The symptoms outlined above will disappear within a short time, but repeated use of the wipe is not recommended in such cases. If, despite treatment, you notice increased inflammatory symptoms (red eyes, the sensation of the presences of a foreign body, tearing, rheumy discharge, pain, or blurred vision) consult an ophthalmologist immediately. Can be used during pregnancy and lactation.

25°C  
2°C

**Store** at room temperature. Keep out of the reach and sight of small children.



#### SolMed Pharma Kft.

19 Ady Endre Street, 1024 Budapest  
Hungary  
Email: [info@solmedpharma.hu](mailto:info@solmedpharma.hu)

[www.ocuvane.com](http://www.ocuvane.com)

